RTM3

CE 0197

- FR Notice d'instructions
- GB Instructions for use
- ES Folleto de instrucciones
- DE Bedienungsanleitung
- IT Avvertenze e istruzioni
- PT Manual de Instruções
- SV Bruksanvisning
- NL Gebruiksaanwijzing
- AR إرشادات الاستخدام

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1. Presentation

The RTM3 and RTM3 DUO are medical devices designed to regulate the flow of a gas. They connect to a wall source of pressurised gases or to a pressure regulator output.

2. Versions

The RTM3 and RTM3 DUO are available in the following versions:

- Gases: Oxygen (O₂) - Medical Air - Carbon dioxide (CO₂) and Carbogen (95% O₂ + 5% CO₂) on request.
- Supply pressure: 4.5 bar (+/- 0.5 bar).
- Connection nozzle: compliant with NF S 90-116 - BS 5682 - DIN 13260 - DISS - U.S. Ohmeda Diamond - NORDIC - UNI ...
- Direct connection or via a rail mounting (tube + crimped tube carrier nozzle + claw).
- Threaded outlet: 12x125 M - 9/16" M - 1/4G M - 1/2" BS F.
- Single or double version.
- Flow ranges:
 - ✓ O₂: 0-1.5 l/min; 0-5 l/min; 0-15 l/min; 0-30 l/min.
 - ✓ Medical Air: 0-5 l/min; 0-15 l/min; 0-30 l/min.
 - ✓ CO₂: 0-12 l/min.
 - ✓ Carbogen: 0-15 l/min.

3. Technical Specifications

- Compliant with NF EN ISO 15002: 2008.
- Measurement unit: litre per minute (l/min).
- Accuracy of measured values: ± 10% of indicated value or ± 0.5 l/min with the highest value being retained.
- Maximum flow rate out of range: 40 l/min.
- Input filters: porosity 35 microns.
- Serial number: with year and month of manufacture and unit number.
- Pressure compensated flow meter (except the 0 to 1.5 l/min flow meter).
- Temperature for use: 23°C.
- RTM3 DUO → 3 positions: nipple outlet, humidifier outlet or OFF position.
- RTM3 DUO not available in 0 to 1.5 l/min.

4. Instructions for use

- Check the condition of the unit.
- Check that the valve is closed.
- Connect the device to the wall socket or to the pressure regulator outlet, in a vertical position.
- For RTM3: Screw on a humidifier if necessary, or a nipple, to link up the medical device tube.
- For RTM3 DUO: Screw a humidifier onto the appropriate output or connect directly to the nipple of the medical device. Select the humidifier output or the nipple output by turning the ring at the bottom of the device.
- Turn the valve on the front to the left until you see the desired flow on the measurement meter scale **(in the middle of the bubble)**.
- For RTM3 DUO: Caution if the valve is not closed, the gas can exit at high throughput when leaving the OFF position.
- Connect the tubing to the patient as the last stage.
- To disconnect: close the valve, remove the tube from the patient, disconnect the humidifier if necessary, and then disconnect the flow meter from the outlet or the pressure regulator outlet.

5. Symbols

	Do not use grease, lubricant or oil
	See instructions
YYMMxxxx	Serial number engraved on the device: YY: Year of manufacture, MM: Month of manufacture, xxxx: Unit number

6. Safety instructions

- To avoid any risk of fire or explosion:
 - ✓ Do not smoke in the vicinity of the device.
 - ✓ Do not use near an ignition point.
 - ✓ Do not place the device in contact with greasy products.
 - ✓ Do not handle the device with greasy hands.
 - ✓ Do not dismantle a pressurised appliance.
- Do not restrict the output of the medical device.
- Influence of pressure: Caution, check the supply pressure to the device. If the inlet pressure is not within the range indicated on the unit, the outflow could waiver from the standard authorised limits.
- Influence of temperature: Please note that temperature variations (between 0°C and 40°C) influence the accuracy of the outflow delivered.
- Do not disassemble the device's inlet or outlet connectors.
- Ensure that the device is always in a vertical position.
- Warning! Do not allow water to enter the flow meter during transport with a humidifier; always unscrew the humidifier before handling.

7. Cleaning and disinfection

- Disconnect the appliance from the gas supply. Clean the outside of the unit with a clean, damp cloth. Allow the device to dry. If using decontaminants, check their compatibility with the plastic (see technical sheet).
- Do not immerse.
- Do not use surface decontaminants.

8. Storage

Store at between -20 and 60°C in a clean, dry place.

If possible, preserve the packaging.

9. Maintenance

RTM3 requires no preventive maintenance during its lifetime.

The device can only be disassembled by a person authorised to do so by Technologie Médicale.

Technologie Médicale recommends an annual inspection of the device:

- ✓ Check the connection to the wall outlet or the pressure regulator outlet.
- ✓ Check that the gas is released correctly at different rates.
- ✓ Check that there is no leakage when the valve is closed.

10. Guarantee

Device guaranteed for 1 year including parts and labour, except for damage or accidents resulting from negligence, inappropriate use, and lack of supervision or maintenance.

Contractual life: 10 years

In accordance with European Directive 93/42/EEC of 14 June 1993, any incident or risk of incident must be reported to TECHNOLOGIE MEDICALE immediately.

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