CE 0197

DEBSON TM2

Débitmètre
Flowmeter
Caudalimetro
Durchflussmesser
Flussimetro
Debitómetro
Debietmeter
Flödesmätare

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1. Presentation

The DEBSON TM2 is a medical device designed to control gas flow. It connects to a pressurised-gas wall fitting or to a pressure regulator output.

2. Versions

DEBSON TM2 is available in the following versions:

- Gas: oxygen (O₂) or medical air.
- Supply pressure: 4.5 bar (+/- 0.5 bar).
- Connector nozzle: compliant with NF S 90-116 - BS 5682 - DIN 13260
US Ohmeda Diamond - NORDIC - UNI...
- Direct connection or via a rail-mounting system (hose + crimped remote probe + rail clamp).
- Threaded outlet: 12x125 M - 9/16" M-1/4G M-1/2" BS F.
- Single or double version.
- Delivered as standard with a stop system.
- Flow ranges (9 flow positions):

	(Flows expressed in l/min)										
0 - 1 l/min	:	0	0.1	0.2	0.3	0.4	0.5	0.6	0.7	0.8	1
0 - 5 l/min	:	0	0.5	0.8	1	1.5	2	2.5	3	4	5
0 - 15 l/min	:	0	1	2	3	4	5	6	9	12	15
0 - 25 l/min	:	0	0.5	1	2	3	4	6	8	15	25
0 - 50 l/min	:	0	5	10	15	20	25	30	35	40	50

3. Technical data

- Compliant with NF EN ISO 15002: 2008.
- Class IIa medical device.
- Measurement unit: litre per minute (l/min).
- Measurement accuracy: $\pm 10\%$ of the value.
- Continuous flow between two positions
- Inlet and outlet filters: porosity 35 μm .
- Individual serial number with month and year of manufacture
- Operating temperature: 23°C.

4. Instructions for use

- Check that the device is in good working order.
- Check that the flow is on 0.
- Connect the device to the wall fitting or the pressure regulator outlet.
- Screw on a humidifier if necessary, or a nipple to link it to the tubing of the therapy equipment.
- Turn the adjustment button or wheel on the front to the left until desired flow is displayed.
- Attach the tubing to the patient at the last stage.
- To disconnect: close the flow meter (position 0 l/min), remove the tubing from the patient, disconnect the humidifier if used, then disconnect the flow meter from the wall fitting or pressure regulator outlet.

5. Symbols

	Do not use grease, lubricant or oil
	See instructions for use
YYMMxxxx	Serial number engraved on the device: YY: Year of manufacture, MM: Month of manufacture, xxxx: Unit number

6. Safety advice

- To avoid any risk of fire or explosion:
 - ✓ Do not smoke in the vicinity.
 - ✓ Do not use near a source of ignition.
 - ✓ Do not allow the device to come into contact with greasy substances.
 - ✓ Do not handle the device with greasy hands.
 - ✓ Do not disassemble a pressurised device.
- Do not constrict the outlet of the medical device.
- Influence of pressure: Caution, check the supply pressure to the device. If the inlet pressure is not within the range shown on the device, there is a risk that the flow may fall outside of the authorised standard range.
- Influence of temperature: Caution, variations in room temperature (between 0°C and 40°C) can alter the accuracy of the flows delivered.
- Do not disassemble the device's inlet or output connectors.
- On this type of device, there is no indicator showing gas flow.
- The flow should be adjusted to one of the click-set positions. Never position the click-set button between two flows.

7. Cleaning and disinfection

- Disconnect the device from the gas supply. Clean the outside of the device with a clean, slightly damp cloth. Leave the device to dry. If using disinfectants, check their compatibility with the plastic (see technical sheet).
- Do not immerse the device.
- Do not use surface disinfectants.

8. Storage

Between -20 and +60°C in a clean and dry place.
If possible, store in original packaging.

9. Maintenance

Preventive maintenance at 10 years.

Only Technologie Médicale authorised personnel can disassemble the device.

Technologie Médicale advises you to carry out a yearly service of the device:

- ✓ Check the connection with the wall fitting or pressure regulator outlet.
- ✓ Check that gas is correctly released at every flow adjustment click setting.
- ✓ Check that there are no leaks when the button is at position 0.

10. Guarantee

Device guaranteed for one year (parts and labour), excluding deterioration or accidents due to negligence, improper use or failure to monitor or maintain the device.

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